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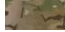
TACTICAL TOURNIQUET CAT

The Tourniquet is an effective tool to help control severe blood loss from significant bodily injuries. The Tourniquet can be used in the line of battle, emergency first aid, and pre-hospital emergency treatment.

Size:	96X3.8cm	89X3.8cm	69X3.8cm		
Color:	Black	Orange	Blue	Tan	Camouflage
					

1. Apply pressure with Hands

Identify where the bleeding is coming from and apply **FIRM** and **STEADY PRESSURE** to the bleeding site. Use two hands if possible.

2. Apply Dressing and Press

Continue to apply **FIRM** and **STEADY PRESSURE** to the bleeding site with dressing or cloth.

3. Apply Tourniquet

If bleeding does not stop, place the tourniquet above the bleeding site. The tourniquet may be applied and secured over clothing.

IF BLEEDING IS SEVER, PLEASE USE THE TOURNIQUET IMMEDIATELY

4. Tactical tourniquet is a single-use device, after use it should be disposed in accordance with the applicable procedures, recommended thermal disposal (incineration).

ZARYS International Group sp. z o.o. sp.k.
ul. Pod Borem 18, 41-808 Zabrze, Poland
www.zarys.com

1 Rev. 00
26.04.2022

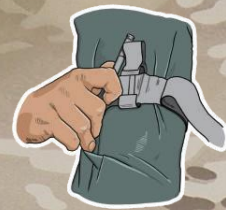
INSTRUCTION FOR USE

1. Insert the limb through the loop of the band or wrap the band around the limb and through the buckle. Position the tourniquet 2-3" above the bleeding site. Pull band tightly.

2. Keep band tight and fasten it back on itself all the way around the wounded limb.

3. Twist the rod until bleeding has stopped.

4. Place the rod inside the clip and lock. Check bleeding conditions and tighten if required reassess if massive bleeding continues.

5. Place the band over the rod and inside the clip. Secure the rod and band with the TIME strap. Record the time of application for future reference.

NOTICE:

- The tourniquet is effective in controlling bleeding for life-threatening bleeding.
- The tourniquet can NOT be applied over solid objects.
- The injured limb should be evaluated, and a second tourniquet could be applied if bleeding cannot be controlled.
- Monitor and provide further treatment on the wound quickly as needed.
- Loosen the tourniquet every 30-60 mins and record the time. Prolonged use can increase the risk of necrosis.
- Store the tourniquet with the band through the buckle with enough slack to slip the tourniquet loop over a limb.

WARNING:

Please read the entire instruction information for application prior to using this product.