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EN FD 312 Surface disinfection

FD 312 is a highly effective, aldehyde-free concentrate for simultaneou disinfection and cleaning of the surfaces of non-invasive medical devices, such as dental chairs, X-ray machines, etc. and of wash-able surfaces and objects such as floors, clinic inventory, walls, etc. FD 312 is gentle on materials. It has a pleasant fragrance.

Composition: 100 g FD 312 contain 6,5 g alkyl-benzyl-dimethyl-ammoniumchloride, 5 - 15 g nonionic tensides, < 5 g alkaline cleansing compounds, < 5 g complexing agents, hexyl cinnamal, linalool, butyl phenyl methyl propional, fragrances and auxiliary agents in aqueous solution.

Effective spectrum: Bactericidal, yeasticidal, limited virucidal activity (encapsulated viruses such as vaccinia viruses, incl. HBV, HCV, HIV¹⁾ as well as non-encapsulated viruses, such as adeno viruses, noro viruses). **List of VAH.** Tested in accordance with EN 13727, EN 13624, EN 14476.

¹⁾ In accordance with the recommendations of the RKI (Robert Koch Institute) (Bundesgesundheitsbl. 47, 62-66, 2004).

Directions for use: FD 312 is used in the form of a 1% solution with a contact time of 15 minutes. For this purpose measure out, for example, 80 ml FD 312 via the measuring beaker in the cap (mark corresponds to 20 ml) and make up to 8 litres with water. For fast disinfection use a 2% strength and 5 min contact time (see Table in DE). FD 312 should not be mixed with ordinary household detergents. Solutions for use should be made up with lukewarm water. We recommend wet mopping using suitable equipment in order to disinfect larger areas. Wet wipe the surfaces of medical equipment. Ensure that even wetting of the treated surfaces takes place. For further information please also see product information on the Internet.

Caution: Put cleaned packaging in the recycling bin.

see imprint.

Danger. Causes serious eye damage. Causes skin irritation. Very toxic to aquatic life. Wear protective gloves and eye/face protection. If skin irritation or rash occurs: Get medical advice/attention. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Rinse skin with water/shower. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point.

DÜRR DENTAL (Products) UK Ltd.
14 Linnwell Way – Telford Way Industrial Estate Kettering, Northants NN16 8PS, United Kingdom ☎ +44 1536526740

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