## DE ID 212 forte Instrumenten-Desinfektion

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  ☞ see bottle imprint.

Danger. Causes severe skin burns and eye damage. Very toxic to aquatic life. Avoid release to the environment. Wear protective gloves and eye/face protection. If skin irritation or rash occurs: Get medical advice/attention. IF SWALLOWED: rinse mouth. Do NOT induce vomiting. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point. Contains LINALOOL. May produce an allergic reaction.

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**Directions for use:** ID 212 forte is used as a 2 % solution. To 20 ml of ID 212 forte add 980 ml of water. The exposition time is in accordance with VAH/DGHM 5 minutes, for TB bacteria 2 % and 60 minutes (see table in DE).

For use in **ultrasonic equipment**, we recommend 2 % for 1 minute, for TB bacteria 4 % and 30 minutes.

Do not use ID 212 forte with ordinary household detergents. Make up the solution for use with lukewarm water. Rinse disinfected instruments in running water, dry and sterilize if necessary. The maximum standing time for the ready-for-use solution is 7 days. When the contamination with blood, saliva etc. is high we recommend renewal of the solution twice or three times per week.

**Caution:** Do not mix ID 212 forte with preparations containing aldehydes, glutaraldehyde or glyoxal, for example. On contact with ID 212 forte, these can cause reddish-brown discolorations. Before using ID 212 forte for the first time, clean the instrument case thoroughly.

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