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EN ID 212 Instrument disinfection

ID 212 is an aldehyde-free concentrate for the disinfection and cleaning of general dental instruments (mirrors, probes, tweezers,

forceps etc.). An additional field of application is the disinfection and cleaning of alcohol- and alcohol-sensitive rotary instruments.

Composition: 100 g ID 212 contain 9 g alkyl-benzyl-dimethyl-ammoniumchloride, 0.1 g guanidine compounds, non-ionic surfactants, alkaline detergents, complexing agents, benzyl salicylate and accessory agents.

Effective spectrum: Bactericidal, tuberculocidal, fungicidal¹⁾, limited virucidal activity²⁾ (encapsulated viruses such as vaccinia viruses, incl. HBV, HCV, HIV as well as non-encapsulated viruses, such as noro viruses³⁾). **List of VAH/DGHM.** Tested in accordance with EN 13727, EN 13624, EN 14476, EN 14561, EN 14562.

¹⁾ Testing with Candida albicans.

²⁾ In accordance with the recommendations of the RKI (Robert Koch Institute)

(Bundesgesundheitsbstl. 47, 62-66, 2004).

³⁾ Test for MNV surrogate virus.

Directions for use: ID 212 is used as a 2% solution. To 20 ml of ID 212 add 980 ml of water. The exposure time is 5 minutes, for TB bacteria 60 minutes (see Table in „DE“). For use in **ultrasonic equipment**, we recommend 2% for 2 minutes, for TB bacteria 2% and 60 minutes.

Do not mix ID 212 with ordinary household detergents. Make up the solution for use with lukewarm water. Rinse disinfected instruments in running water, dry and sterilize, if necessary. The maximum standing time for the ready-for-use solution is 7 days. When the contamination with blood, saliva etc. is high we recommend renewal of the solution twice or three times per week.

☞ ☞ see bottle imprint.

Warning. Causes serious eye irritation. Causes skin irritation. Very toxic to aquatic life. Wear protective gloves and eye/face protection. If skin irritation or rash occurs: Get medical advice/attention. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Rinse skin with water/shower. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point.

DÜRR DENTAL (Products) UK Ltd. 14 Linnwell Way - Telford Way Industrial Estate Kettering, Northants NN16 8PS, United Kingdom ☎ +44 1536526740

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