Sectional  
matrix  
systems

# Quickmat<sup>®</sup> deluxe

en - ENGLISH

Sectional matrix systems

All individual components are CE marked.  
Medical Devices Kit for professional use only

## Instructions for use

Description: Quickmat Deluxe is a sectional matrix system leading to precise and anatomically correct contact points in the restoration of class II cavities. This kit features Polydental's Quickring, a classic sectional matrix ring made of stainless steel, and the insert "Re-Force" which can be added to the ring if there is a clinical need to increase the separation force in combination with the included Interdental Maple Wood Wedges. For the restoration of very extensive cavities, the kit also includes Silicone Tubes and Delta Tubes. These are angular wedges that allow silicone wedges to be used to "entfernen" and "entfernen" the margins of the ring. Hintschlinge sectional matrices complete the kit's assortment. **Composition:** Quickring: stainless steel, Insert "Re-Force": thermoplastic. Quickmat: stainless steel, Silicone Tube: silicone, Delta Tube: silicone, Wood wedge: maple wood. **Indications:** Quickmat Deluxe Kit is indicated for gentle tooth separation in order to create correct contact points in the restoration of class II cavities. **Contraindications:** None known. **Side Effects:** There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydental.

## Interactions with other medical devices or products:

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## Komposition:

Quickring: stainless steel, Insert "Re-Force": thermoplastic. Quickmat: stainless steel, Silicone Tube: silicone, Delta Tube: silicone, Wood wedge: maple wood. **Indications:** Quickmat Deluxe Kit ist indicated for gentle tooth separation in order to create correct contact points in the restoration of class II cavities. **Contraindications:** None known. **Side Effects:** There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydental.

## Interactions with other medical devices or products:

There are no known interactions.

## Information for the patient:

No information

## Treatment:

Self-treatment

## Treatment: