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 Little Doctor Electronic (Nantong) Co. Ltd., No.8, Tongxing Road Economic & Technical Development Area, 226010 Nantong, Jiangsu, People's Republic of China

 LITTLE DOCTOR EUROPE SP. Z O. O., 576 Zawila Street Krakow 30-390 Poland.

© Registered Trade marks of Little Doctor International (S) Pte. Ltd.  
© Little Doctor International (S) Pte. Ltd., 2000-2017

B084/1712/14

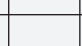
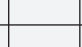
 DOWNLOAD USER MANUAL  
[www.littledoctor.sg/manuals](http://www.littledoctor.sg/manuals)

**BGR** Маншет LD-CUFF.  
**CZE** Manžeta LD-CUFF  
**ENG** Blood pressure LD-CUFF  
**HUN** LD-CUFF mandzsetta  
**KAZ** LD-CUFF көмкермесі

**POL** Mankiet LD-CUFF  
**ROU** Manșeta LD-CUFF  
**RUS** Манжета LD-CUFF  
**UKR** Манжета LD-CUFF

								
WITHOUT RING	7-12 cm		<b>N</b> NEONATAL	COTTON	<b>C1N</b>	<b>C2N</b>		
	11-19 cm		<b>I</b> INFANT	COTTON	<b>C1I</b>	<b>C2I</b>		
	18-26 cm		<b>C</b> CHILD	COTTON	<b>C1C</b>	<b>C2C</b>		
				NYLON	<b>N1C</b>	<b>N2C</b>		
	25-40 cm		<b>A</b> ADULT	COTTON	<b>C1A</b>	<b>C2A</b>		
NYLON				<b>N1A</b>	<b>N2A</b>			
34-51 cm		<b>L</b> LARGE ADULT	COTTON	<b>C1L</b>	<b>C2L</b>			
			NYLON	<b>N1L</b>	<b>N2L</b>			
D-RING		<b>T</b> THIGHT	COTTON	<b>C1T</b>	<b>C2T</b>			
			25-36 cm		<b>A</b> ADULT	NYLON	<b>N1AR</b>	<b>N2AR</b>
						33-46 cm		<b>L</b> LARGE ADULT

**Little Doctor International (S) Pte. Ltd.**

Yushun Central P.O. Box 9293, Singapore 917699 Fax: (65) 62342197 E-mail: info@littledoctor.sg

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**ENG** Cuff model LD-CUFF for LD-series blood pressure monitors. Range of measurement of pressure 0-300 mmHg. Conditions of operation: temperature +10°C +40°C; humidity 85% or below. Conditions of transportation/storage: temperature - 34 °C + 65 °C; humidity 85% or below. Protect the bladder and tubes from sharp objects. The cuff is resistant to multiple sanitization. Use only a soft, dry cloth to clean the unit. Do not use gasoline, paint thinner, or other strong solvents on the unit. Since the cuff may absorb perspiration and other fluids, inspect it

for stain and discoloration after each use. When cleaning the cuff, do not machine wash or scrub it. Use a synthetic detergent and gently rub the surface. Warranty time 6 months from the date of purchasing. Operating life 3 years since transfer to the consumer.. After the expiration of the term lifetime is necessary periodically apply to specialized service centres to verify the possibility of further use. During recycling follow your local rules. The producer does not prescribe the special conditions of recycling for this device. Year of manufacture indicated on the label. Product of Little Doctor International (S) Pte. Ltd., Yishun Central P.O. Box 9293 Singapore 917699. Distributor in Europe: Little Doctor Europe Sp. z o.o. 57G Zawila Street, 30-390, Kraków, Poland. Sales Office phone: +48 12 2684746, 12 2684747, fax: +48 12 268 47 53. E-mail: biuro@littledoctor.pl www.LittleDoctor.pl

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